

API Pharma USA (based in Miami, FL) offers a full array of laboratory diagnostics for COVID-19.

Our **antigen tests**, are perfect for those who want to test their employees, residents, students or any other at risk population.

Sensitivity: 92.5% Specificity: 99.2%
 Compared to PCR deep nasopharyngeal swabs, antigen superficial nasal swabs have

- Reliable results in 20 minutes
- Works with nasal, nasopharyngeal, or oropharyngeal swabs
- Swabs and reagents all included
- Doesn't require a lab or specially-trained personnel
- CE Mark approved (European Union)

Covid-Rapid Antigen Test



Proudly Made in the USA

Simple Directions

Covid-Rapid Antigen Test

Step 1

Collect sample in any one of three ways



Nasopharyngeal swab



Throat swab




Nasal swab

Step 2

Place in tube with 6 drops of buffer


Swirl for 10 seconds

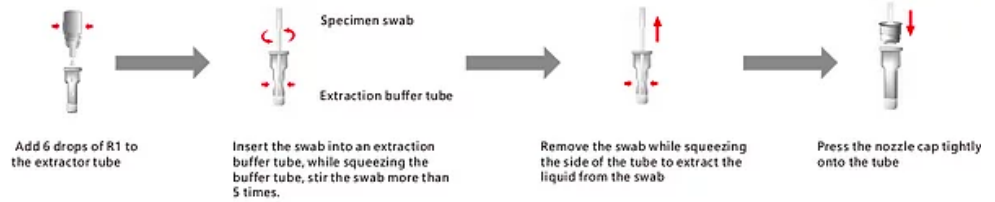

Step 3

Wring out swab while pulling out and place cap on table

Add 2 drops of reagent with sample to card

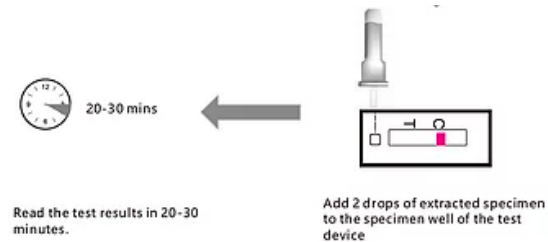

Step 4

Interpret results after 20 minutes

